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Dipping lacquer for silicone no skin-formation

Application: For all silicone based impressions & earmoulds. Prior to application (dipping) the earmould must be absolutely clean, dry and free of grease! Hint: Use an appropriate surface cleaner (e.g. isopropyl alcohol). After shaking-off the surplus of lacquer, let the impression resp. the earmould (depending on coating thickness and humidity) dry for approx. 10 min at room temperature 23 °C/ 73 °F. The surface of the impression/earmould shows now a glossy layer. Apply lacquered otoplastic only when completely odorless

Important: Clean bottle edge immediately after use, then close bottle tightly! To ensure the quality of dip coat, do not return residues of lacquer back into original bottle. Do not store in the fridge.

Please note: During application of lacquer use a suction system!
For use by trained specialists.
Please refer to the safety data sheet!

Safety advice

On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

Caution: Federal U.S. law restricts this device to sale by or on the order of a licensed healthcare practitioner (or trained specialist personell).

Made in Germany by

DETAX

Carl-Zeiss-Str. 4
76275 Ettlingen
www.detax.de

CE 0463

15 °C
59 °F

25 °C
77 °F

03/2018