micropor

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Silicone lacquer for surface smoothing and sealing of silicone based earmoulds and impressions, smooth coating – heat curing

Application: For all silicone based impressions & earmoulds. Prior to application (dipping/brushing) the impression must be absolutely clean, dry and free of grease!

Hint: Use an appropriate surface cleaner (eg. isopropyl alcohol). After shaking-off the surplus of lacquer, let the impression resp. the earmould (depending on coating thickness and humidity) dry in the oven at 100 °C/212 °F for approx. 15 min. The surface of the impression resp. earmould shows now a glossy layer.

Caution: Avoid contact with eyes/skin. Wear protective gloves/eye protection. Do not breathe vapours.

Important: Clean bottle edge immediately after use, then close bottle tightly! To ensure the quality of micropor, do not return residues of lacquer back into original bottle. Do not store in the fridge.

Please note: During application of lacquer use a suction system!

Safety advice:


On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

For use by trained specialists.

Please refer to the safety data sheet!

Made in Germany by

DETAX Carl-Zeiss-Str. 4
76275 Ettlingen
www.detax.de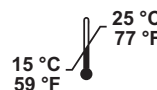

03/2018