# softwear coat

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## Antibacterial lacquer for equalizing of generative manufactured silicone earmoulds

**Application:** For all silicone earmoulds. Prior to application (dipping/brushing) the earmould must be absolutely clean, dry and free of grease! Hint: Use an appropriate surface cleaner (e.g. isopropyl alcohol). After shaking-off the surplus of lacquer, let the earmould (depending on coating thickness and humidity) dry at 80 °C/176 °F for approx. 20 min. or at room temperature 23 °C/73 °F over night. The surface of the earmould shows now a glossy layer. Apply lacquered otoplastic only when completely odorless.

**Caution:** Avoid contact with eyes/skin. Wear protective gloves/eye protection. Do not breathe vapours.

**Important:** Clean bottle edge immediately after use, then close bottle tightly! To ensure the quality of softwear coat, do not return residues of lacquer back into original bottle. Do not store in the fridge.

**Please note:** During application of lacquer use a suction device!

**Safety advice:** On condition of a proper application, undesired effects are not to be expected. However, reactions of the immune system like allergies, irritations, cannot be absolutely excluded. In case of doubt, we recommend to make an allergy test before the application of the material.

For use by trained specialists.  
Please refer to the safety data sheet!

Caution: Federal U.S. law restricts this device to sale by or on the order of a licensed healthcare practitioner (or trained specialist personell).

Made in Germany by

**DETAX**

Carl-Zeiss-Str. 4  
76275 Ettlingen  
www.detax.de

15 °C  
59 °F

03/2018